

Lamp Magnifier Stand Assembly Instructions (#GL-715000)

(Stand attaches to Lamp Magnifiers #2772 and #2777)

May 2013

①



Contents of Lamp Magnifier Stand box (5-pronged base, pole, bag of 5 casters, metal end-piece, thin plastic ring, thick plastic ring with notch, plastic end-cap).

②



Snap the 5 casters into the base.

③



Screw the pole into the base. Use the end of the pole that has the nut around it just above the threads of the pole.

④




Tighten the nut at the bottom of the pole.

⑤



Lamp Magnifier (purchased separately) with 4 pieces to attach to the bottom of the Lamp so that it attaches to the stand.

⑥

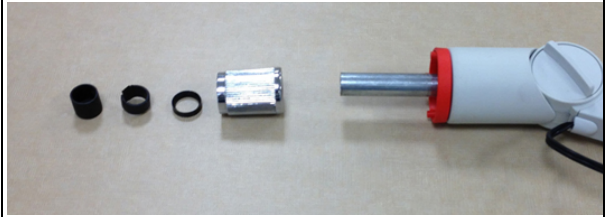

The 4 pieces to attach (left to right): plastic end-cap, thick ring (with notch in it), thin ring (no notch), metal end-piece

⑦



Slide the metal end-piece over bottom of Lamp Magnifier arm, smaller hole towards the Lamp Magnifier

⑧



Slide on the thin ring (no notch)

⑨



Slide on the thick ring (with notch)

⑩



Push on the plastic end-cap

⑪



Complete Lamp Magnifier with all pieces attached

⑫



Place the Lamp Magnifier on top of metal pole that is from the base and tighten

⑬



Complete Lamp Magnifier on moveable stand