# mobilux **DIGITAL**

Bedienungsanleitung  
User Manual  
Mode d'emploi  
Istruzioni d'uso  
Manual de instrucciones  
Gebruiksaanwijzing  
Betjeningsvejledning  
Brugsanvisning  
Bruksanvisning  
Käyttöopas  
Instrukcja obsługi  
Provozní návod  
サービスマニュアル

**ESCHENBACH**  
Germany

Deutsch .....	4
English.....	7
Français .....	10
Italiano.....	13
Español.....	16
Nederlands.....	20
Dansk .....	23
Svenska.....	26
Norsk.....	29
Suomi.....	32
Polski.....	35
Česky .....	39
日本語.....	42

## Deutsch

### Einleitung

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## English

### Introduction

You have acquired a brand name product from Eschenbach that has been manufactured according to the most modern production processes. We congratulate you on your decision.

Before initial operation you should read through the instructions carefully and become acquainted with all of the device's functions. Mobilux digital is intended for use as an electronic visual aid in as much as it presents a magnified image on the screen.

### Safety Instructions

- ▶ Danger of glare and injury! Do not look directly at the light source on the underside of the housing!
- ▶ Do not allow children to handle electronic devices without supervision.
- ▶ Check the device and the mains adapter with power cable for visible damage. If the device, the mains adapter or its cable are damaged immediately disconnect the mains adapter from the mains power. Have the product checked by a qualified specialist before you use it again.

- ▶ Only plug the mains adapter into an approved installed mains power socket whose mains voltage corresponds to the technical data given in these instructions.
- ▶ Use, exclusively, the supplied mains adapter for operation of the mobilux digital.
- ▶ Never open the mobilux digital. There are no serviceable parts whatsoever inside. In these cases safety can not be assumed and the warranty becomes invalid.
- ▶ Never expose the mobilux digital to moisture.
- ▶ Always grasp the mains adapter directly and never with wet hands, since this could cause a short circuit or electrical shock, to pull it out of the mains socket.
- ▶ Never tie knots in the cable, do not bind it together with other cables and make sure that it is not trapped.
- ▶ Risk of tripping! Make sure that the mains adapter cable does not pose a danger.
- ▶ Protect the mobilux digital from knocks or blows and excessive heat! Never place the mobilux digital on heating devices or in the sun!
- ▶ Inform other persons, particularly children, of these dangers!

## Initial device operation

Mobilux digital has an integrated lithium ion rechargeable battery.

**Before using for the first time the rechargeable battery must be fully charged.** Charging takes about 2 hours. During the charging process the red LED (2) over the power supply socket (1) lights up. The LED goes out when the rechargeable battery is fully charged. You can use the mobilux digital while it is charging.

Use the mains adapter, with the customary plug adapter for your country (EU, GB, US), only in a properly connected mains socket. To change the plug adapter slide the locking mechanism on the mains adapter in the direction of the arrow and remove the plug adapter. Insert the desired plug adapter and release the locking mechanism.

A loop (6) is provided at the end of the handle to facilitate the hanging of the mobilux digital. A lanyard as well as a cleaning cloth and case are enclosed.

**Switching on/off**  
Slide the on/off switch (3) in the direction of the display (4) to turn the device on. The LEDs on the underside of the device will begin to light up and in the display the charging condition of the rechargeable battery is shown briefly. The indicator appears blue with mains power operation and green with rechargeable battery operation.

## Switching on/off

Slide the on/off switch (3) in the direction of the display (4) to turn the device on. The LEDs on the underside of the device will begin to light up and in the display the charging condition of the rechargeable battery is shown briefly. The indicator appears blue with mains power operation and green with rechargeable battery operation.

Hold the mobilux digital with the display centred over the text that is to be reproduced, magnified, in the display.

## Selecting the magnification/display modes

With the ↻ button (8) you can switch between 3-fold and 6-fold magnification.

Press the Ⓚ button (7) to switch between the display modes (true colour display as well as contrast enhanced positive and negative displays).

When switched on the previously set mode (magnification, colour) is maintained.

## Switching illumination on/off

If you hold down the Ⓚ button (7) for 3 seconds the LED illumination switches off. This is useful when reading with reflective backgrounds, for example, road maps behind glass or high gloss brochures.

## Adjusting for fixed-images

If you press the 📷 button (5) the image currently depicted in the display will freeze. Thus you can, for example, display text that is located in a poorly accessible spot.

## Automatic switch off

After 3 minutes without image changes the mobilux digital switches to the energy saving mode and the display becomes black.

With movement the display is automatically switched back on. After 8 minutes in the energy saving mode the mobilux digital automatically switches off. To switch the device back on press one of the buttons or turn the on/off switch (3) off and then back on again. If the device functions fail the built-in rechargeable battery is depleted. Power supply can be restored to the device with the enclosed mains power adapter attached to the mains socket.

### Care Instructions

- ▶ When cleaning the mobilux digital do not use any soap solutions that contain softening agents, no alcoholic or organic solvents and no abrasive detergents! The lenses could be destroyed!
- ▶ Do not clean the mobilux digital in an ultrasonic bath or under flowing water!
- ▶ Clean the mobilux digital with the enclosed cleaning cloth. For heavier soiling (e.g. fingerprints), moisten the cloth slightly.

### Technical Data

Supply voltage:	100-240 V~, 50-60-Hz
Display:	3.5", 16:9
Working temperature:	-20°C ~ +70°C
Rechargeable battery charging time:	approx. 2 h

Rechargeable battery runtime: True colour mode: approx. 3 h  
Positive mode: approx. 2 h  
Negative mode: approx. 4 h  
162 g.

Weight:

### Warranty

Within the parameters of the applicable statutory provisions, we provide a warranty for the functioning of the product described in this manual with regard to faults arising in connection with manufacturing defects or material defects. In event of damage due to improper handling, including damage from a fall or impact, no warranty claims can be made. Warranty claims are only accepted upon presentation of the sales receipt!

### Disposal

- ▶ When the device reaches the end of its useful life, do not dispose of it with the normal household waste. This product is subject to the European directive 2002/96/EC.
- ▶ Dispose of the product or parts of the product through a certified disposal facility or your municipal disposal facility. Observe the currently applicable regulations. In case of doubt, contact your disposal facility. Dispose of all packaging materials in an environmentally friendly manner.

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Eschenbach Optik GmbH