system varioPLUS

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English

You have purchased a quality product from the Eschenbach company that was "Made in Germany" using state-of-the-art manufacturing processes. We congratulate you on your decision.

For the illuminated magnifier system vario plus, various magnifications and different lighting options are available. The illuminated magnifier acquired by you has been specifically assembled with this system according to your visual requirements. It may therefore differ from the illustration.

Should there be changes in your visual requirement, the lighting and magnification can be variably adjusted by changing the magnifier handle and magnifier pot.

The lens of your illuminated magnifier is made of high quality, unbreakable PXM[®] material, with a cera-tec[®] hard coating protecting it against scratches - scratch resistance is a crucial criterion for quality and durability.

Safety Instructions

- ▶ **Danger of fire!**
Lenses in optical devices can cause significant damage by focusing light to generate heat if handled or stored improperly. Be careful never to leave optical lenses lying in direct sunlight!
- ▶ **Danger of glare and injury!**
Never look directly into the sun or other bright light sources using optical devices! 159960: Never look directly into the light source.
- ▶ **Protect your magnifying light from impacts, moisture and excess heat.**
Never lay your magnifying light on a radiator or in direct sunlight.
- ▶ **Risk of tripping! Make certain that the power cord does not pose a risk!**
- ▶ **Do not wind the power cord around the magnifying light.**
- ▶ **Do not use the magnifying light in the vicinity of water-filled containers (e.g. bath tub)!**
- ▶ **Inform other persons, particularly children, of these dangers.**

Notes on usage

- ▶ **Connect the mains adapter ❸ to a power socket.**
-  A slight warming of the mains device and lamp during operation is normal.
 The illuminated magnifier can be left plugged in when not in use (switched off).
- ▶ **When the lens is not standing horizontal, pull the foot ❶ from the handle until it engages.**
- ▶ **If the foot bothers you, press the snap-in pin ❷ and slide the foot into the handle.**
-  The distance between the lens and the text to be read is optimal when the illuminated magnifier is lying on the page.
- ▶ **Switch the illuminated magnifier on by sliding the switch ❹ in the direction of the lens.**
- ▶ **Now place the illuminated magnifier on the text to be read.**
- ▶ **Guide the illuminated magnifier with the handle ❺ over the text to be read.**
- ▶ **After use, switch the illuminated magnifier off by sliding the switch ❺ to the rear.**

If you do not like the colour of the light, your optician can adjust the light colour with the supplied coloured LED-Caps ❻.

If the light is too strong, the brightness can be reduced by using the switch ❹.

Magnifier pots with valuable extra function

Pivoting reading line ⑦ as an orientation guide

The following magnifier pots have this function:

| ø 80, 12 Diopter, 3.0 x | ø 70, 16 Diopter, 4.0 x | ø 58, 20 Diopter, 5.0 x |

► Turn the lever ⑧ to swing in the orientation guide.

Pivotable additional magnification ⑨ of 4 Diopter.

To read small print or to highlight difficult passages of text.

The following magnifier pots have this function:

| 100x50, 7.6 Diopter, 3.0 x | 100x75, 7.0 Diopter, 2.8 x |

► Turn the lever ⑩ to swing in the additional magnification.

Care Instructions

 When the illuminated magnifier is not in use, it should be covered.

Clean the housing with a moist cloth. Clean the lenses with a soft, lint-free cloth (e.g. eyeglass cleaning cloth).

In the event of extreme soiling (e.g. fingerprints): Pull the magnifier head off the handle and clean the lens with a damp cleaning cloth. When it is completely dry, reassemble the front part and the handle.

Do NOT use any soap solutions that contain softening agents, alcohol-based solvents or abrasive cleansers! The lenses could be damaged.

Warranty

Within the framework of the applicable statutory provisions, we provide a warranty for the functioning of the product described in this manual with regard to faults arising in connection with manufacturing errors or material errors. In event of damage due to improper handling, including damage from a fall or impact, no warranty claim exists. Warranty claims are only accepted upon presentation of the sale receipt!

Disposal

When the device reaches the end of its useful life, do not dispose of it with normal household waste. Dispose of the product or parts of the product through a certified disposal agency or your municipal disposal agency. Observe the currently applicable regulations. In case of doubt, contact your disposal agency. Dispose of all packaging materials in an environmentally friendly manner.

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Eschenbach Optik GmbH