

maxiPLUS economy

Bedienungsanleitung
Operating instructions
Mode d'emploi
Istruzioni d'uso
Instrucciones de uso
Gebruiksaanwijzing
Brugsanvisning
Bruksanvisning
Käyttöohje
Návod k použití
Instrukcja obsługi
サービスマニュアル

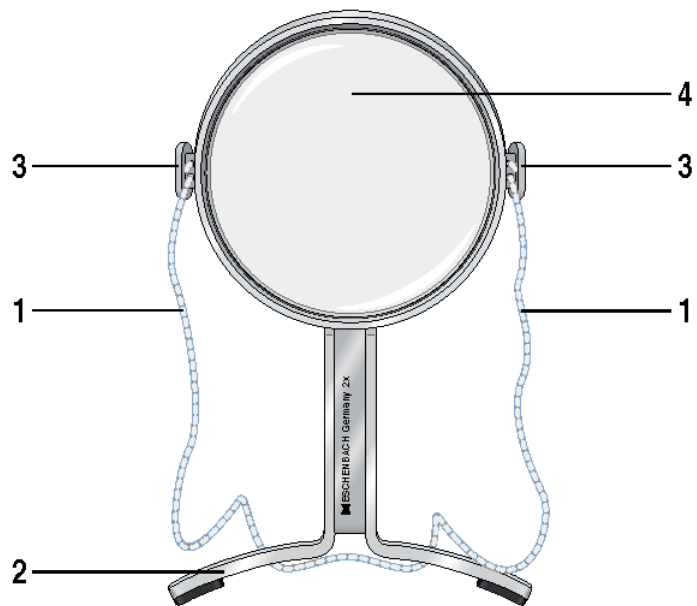 **ESCHENBACH**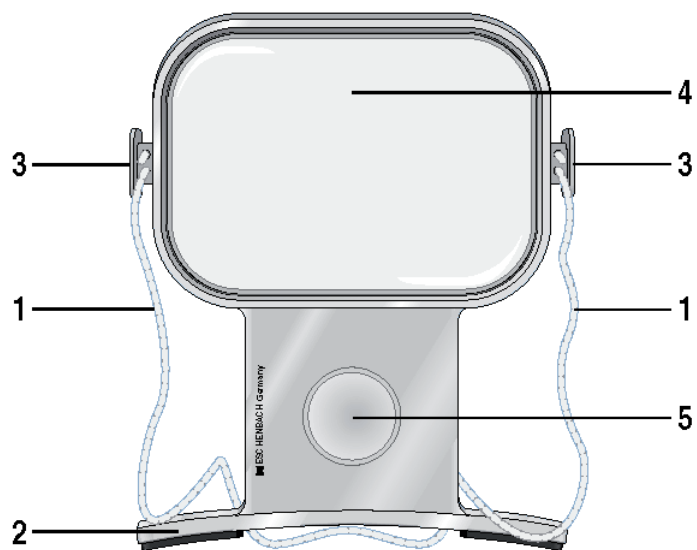

Deutsch 4
English 6
Français 8
Italiano 11
Español 13
Nederlands 15
Dansk 18
Svenska 20
Norsk 22
Suomi 24
Polski 27
Česki 29
日本語 31

Deutsch

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English

Dear Customer:

Congratulations on purchasing this Eschenbach hands-free magnifying glass, a quality product "Made in Germany". We hope you enjoy using it every day.

This quality Eschenbach hands-free magnifying glass is a visual aid for everyday situations, such as crocheting, knitting, sewing, embroidering and threading needles as well as modelling and many other hobbies and even nail care.

Whatever you are doing, it leaves both hands free to work with.

As market leader for magnifying glasses and reading glasses, we at Eschenbach can look back on many years of experience, which is reflected in the quality, handling and design of our products.

Safety instructions

- **Risk of fire!**
Lenses used in optical equipment may cause considerable damage by their "burning glass effect" if

used or stored incorrectly. Ensure that optical lenses are never left uncovered in the sunlight!

- **Danger of blinding or injury!**
Never look at the sun through any optical equipment.
- **Protect your hands-free magnifying glass from knocks and impacts, moisture and excessive heat. Never leave your magnifying glass on a radiator or in the sun.**
- **Make sure others, especially children, are aware of this danger!**

Care tips

Keep the magnifying glass covered when you are not using it.

Clean the lenses with a soft, lint-free cloth, such as a cloth for cleaning glasses. If they are very dirty (for example with fingerprints), clean the lenses with lukewarm water.

- **Do not use any soapy solutions that contain softeners, alcohol-based solvents, or abrasive cleansers.**
This may damage the lenses beyond repair.

Instructions for use

- ▶ Hang the cord [1] of the hands-free magnifying glass around your neck.
- ▶ Place the curved support [2] of the magnifying glass on your chest or stomach so that the magnifying glass is at a right angle to your body.
- ▶ By winding the cord [1] around the cord holders [3] on the sides of the magnifying glass, you can adjust the working distance as needed.

The maxi plus 2678 hands-free magnifying glass has two lenses.

The large lens [4] (140x100 mm), which magnifies to twice normal size, is for various craft and hobby tasks, and the

small lens [5] (Ø 35 mm) is suitable for tasks requiring higher precision, such as threading a needle.

The economy 2677 hands-free magnifying glass has one lens.

Technical data

Item	Lens size	dpt	Magnification
maxi plus 2678	100 x 140 mm	2,6	2 x
	Ø 35 mm	11,2	4 x
economy 2677	Ø 100 mm	3,2	2 x

Do you need more magnifying power?

Many more useful Eschenbach visual aids are available from your optician.

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Garantee

We are required by law to guarantee the function of the product described in these descriptions in respect of defects caused by manufacturing or material faults. Any damages caused by incorrect handling, including damage caused by being dropped or knocked, are not covered under the guarantee. Guarantee benefits can only be claimed through proof of purchase in the form of a sales receipt!

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Eschenbach Optik GmbH