# easyPOCKET

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**Operating instructions**

**Mode d'emploi**

**Istruzioni d'uso**

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**Brugsanvisning**

**Bruksanvisning**

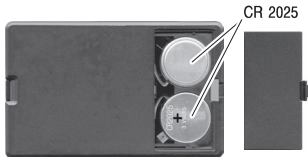
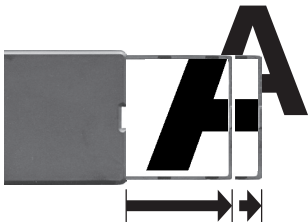
**Käyttöohje**

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 **ESCHENBACH**

CE

## Deutsch

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## English

### **Safety instructions**

- **Danger of fire! Lenses used in  
optical equipment may cause  
considerable damage through  
their 'burning glass effect' if  
used or stored incorrectly. En-**

**sure that optical lenses are never left uncovered in the sunlight!**

- **Danger of blinding or injury Never look at the sun through any optical equipment! Do not look directly at the light sources! Risk group 1 as per EN 62471:2008. Danger value in case of exposure (EHV): 2200 seconds in 800 mm. The free group limit values are complied with at a distance of 1.8 m. Retinal danger due to actinic light 400 nm to 780 nm.**
- **Protect your magnifier from knocks, bangs, moisture and excessive heat. Never place your magnifier on heaters or in the sunlight.**
- **Ensure that other people, especially children, are aware of these dangers!**

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## **Guarantee**

We are required by law to guarantee the function of the product described in these descriptions in respect of defects caused by manufacturing or material faults. Any damages caused by incorrect handling, including damage caused by being dropped or knocked, are not covered under the guarantee. Guarantee benefits can only be claimed through proof of purchase in the form of a sales receipt!

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Eschenbach Optik GmbH