# headlight LED

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User Manual

Mode d'emploi

Istruzioni d'uso

Manual de instrucciones

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Bruksanvisning

Käyttöopas

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CE

 ESCHENBACH

Deutsch .....	3
English.....	14
Français .....	24
Italiano.....	35
Español.....	46
Nederlands .....	56
Dansk .....	66
Svenska.....	76
Norsk.....	86
Suomi.....	96
Polski.....	106
Česky .....	116
日本語.....	126

## Deutsch

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## English

1. Cable retainer for MaxDetail
2. Clip for combiPLUS
3. Clip for maxDETAIL
4. Clip for laboCOMFORT
5. headlight LED
6. Attachable belt clip
7. Cable guides
8. Connecting cable
9. On/Off switch
10. Battery compartment cover
11. Retainer for connecting cable

Dear Customer,

we would like to congratulate you on the purchase of this headlight LED – a top-quality product from Eschenbach – and hope you enjoy using the device on a daily basis.

As the market leader for magnifying glasses and reading glasses, we at Eschenbach are able to look back on many years of experience which are mirrored in the quality, handling and design of our products.

### **Before initial use**

Before using the device for the first time, please read the entire manual carefully so that you can get to know all the functions of the device.

## Safety instructions

- ▶ **Risk of dazzling and injury! Do not look at the light source on the underside of the housing for long periods! Risk group 2 in accordance with EN 62471:2008. Exposure hazard value EHV = 60. The risk group 1 limit values are complied with at a distance of 40 cm, the free group limit values are complied with at a distance of 66 cm. Retina hazard due to blue light 400 nm to 780 nm.**
- ▶ **Take care not to dazzle other people or animals during use.**
- ▶ **Children may not handle electronic devices without supervision.**
- ▶ **If your headlight LED or the cable exhibit any visible damage, have the product checked and, if necessary, repaired by a qualified technician.**
- ▶ **Never open the housing of the headlight LED! There are no serviceable parts whatsoever inside. The guarantee will be void if the device is opened.**

- ▶ **Never expose the headlight LED to moisture.**
- ▶ **Protect the headlight LED from knocks or blows and excessive heat! Never place the headlight LED on a heater or in direct sunlight!**
- ▶ **Ensure that other people, particularly children, are aware of these risks!**

## **Package contents**

- ▶ Headlight LED incl. battery case
- ▶ 3 × Industrial batteries (Type AA / 1.5V / not long life)
- ▶ Belt clip
- ▶ Case
- ▶ Operating instructions

Depending on the item number of your product, the following accessories are also supplied:

### Item no. 16042:

- ▶ 2 rubber clips for the cable guide on the temple (⑦)
- ▶ 1 cable clip for the cable guide on the maxDETAIL temple (①)
- ▶ Adapter for combiPLUS (②), maxDETAIL (③) and laboCOMFORT (④)

### Item no. 160421:

- ▶ 2 rubber clips for the cable guide on the temple (⑦)
- ▶ Carrier frame

### Item no. 160422:

- ▶ 2 rubber clips for the cable guide on the temple (⑦)
- ▶ Clip for attachment to a spectacle frame

### Item no. 162452:

- ▶ Cable clip for the cable guide on the temple (①)
- ▶ maxDETAIL

## Assembly

The adapter for combiPLUS (2) is inserted from below on the frame edge.

The adapter for maxDETAIL (3) is attached to the bridge of the centre section.

The adapter for laboCOMFORT (4) is inserted in the cut-out in the headband.

For articles 160421 and 160422, the carrier mounting and the clip are supplied with the adapter. On delivery, the light element of the head-

light LED is already clipped into the adapter. With item 162452, the cable clamp (❶) is also already clipped onto the temple.

To open the battery compartment cover (❷) push it down and insert  $3 \times 1.5$  V size 'AA' batteries in accordance with the polarity diagram in the battery compartment. We recommend the use of rechargeable batteries for frequent use. Replace the battery compartment cover.

Attach the belt clip to the back of the battery case for extra mobility. You can also put the battery case into a pocket of your clothing.

Two separate rubber clips (❸) are supplied in varying thickness for use as a cable guide on temples of differing thicknesses. These ensure a clean, trouble-free routing of the cable along the temple. Attach the appropriate cable clip for the temple of your glasses and press the cable into the groove:

## Operation

Press the on/off button (9) once to turn on the light. Press this button again to turn the light to a lower level (Eco mode). Press it again to turn off the headlight LED. If the battery power becomes very low, the light starts to flicker. Switching over from full to Eco-mode is no longer possible.

On the back of the battery case you will find a cable retainer (11) for storing the connecting cable (8) when not in use.

## Technical details

Batteries:	3 × 1.5 V AA batteries
Battery recommendation:	AA long life batteries or rechargeable batteries
Operating time:	approx. 8 hours in full mode
<small>(when using long life batteries)</small>	approx. 13 hours in Eco mode
Temperature range:	0°C–50°C
Weight of lighting element:	8 g (without clip or adapter)

## Care Instructions

- ▶ When cleaning the headlight LED do not use any soap solutions that contain softening agents, alcoholic or organic solvents, and do not use abrasive detergents!
- ▶ Do not clean the headlight LED in an ultrasonic bath or under flowing water!

- ▶ Only clean the headlight LED with a soft, dry cloth. For heavier soiling (e.g. fingerprints), moisten the cloth slightly.

## **Warranty**

Within the parameters of the applicable statutory provisions, we provide a warranty for the functioning of the product described in this manual with regard to faults arising in connection with manufacturing defects or material defects. In event of damage due to improper handling, including damage from a fall or impact, no warranty claims can be made. Warranty claims are only accepted upon presentation of the sales receipt!

## **Disposal**

- ▶ When the device reaches the end of its useful life, do not dispose of it with the normal household waste. This product is subject to the European directive 2012/19/EC.

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Eschenbach Optik GmbH