maxDETAIL Clip

Bedienungsanleitung
User Manual
Mode d'emploi
Istruzioni d'uso Manual
de instrucciones
Gebruiksaanwijzing
Betjeningsvejledning
Bruksanvisning
Bruksanvisning
Käyttöopas
Instrukcja obsługi
Provozní návod
サービスマニュアル

ESCHENBACH

Deutsch	3
English.....	7
Français	11
Italiano	16
Español.....	20
Nederlands.....	24
Dansk	28
Svenska.....	32
Norsk.....	36
Suomi.....	40
Polski.....	44
Česky	48
日本語.....	52

Deutsch

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English

You have purchased a quality product from the Eschenbach company that was “Made in Germany” using state-of-the-art manufacturing processes. We congratulate you on your decision.

MaxDetail Clip is a magnifying attachment for clipping onto existing eyeglasses. This allows you to see twice as large without straining your eyes. Crossword puzzles and working within arm’s reach, whether at home or at work, are easy thanks to the double magnification. Modern design and effective, easy use were the main priorities during development of this product.

Safety Information

- ▶ Danger of glare and injury! Never look directly into the sun or other bright light sources using optical devices!
- ▶ Danger of fire!
Lenses in optical devices can cause significant damage by focusing light to generate heat if handled or stored improperly. Be careful never to leave optical lenses lying in direct sunlight!

- ▶ Also inform other people and children especially of this safety measure!
- ▶ MaxDetail Clip may not be used while driving automobiles.
- ▶ Protect your “MaxDetail Clip” from impacts and excessive heat. Never lay your “MaxDetail Clip” on a radiator or in direct sunlight.
- ▶ Be particularly careful when using the product on eyeglasses with plastic lenses. When used improperly, the eyeglasses may be scratched since the lens of the product rests directly against the lens of the eyeglasses.

Adjustment

The lens of the “MaxDetail Clip” is designed for a working distance of about 400 mm from the eye.

To attach the “MaxDetail Clip” to your eyeglasses, press the clip together.

When you wish to stop using the “MaxDetail Clip”, simply flip the clip up, it is not necessary to remove it.

To allow an optimal line of sight on both large and small eyeglasses, the lens assembly on the clip can be moved vertically. To do this, it is necessary to unscrew the lens assembly with a small Philips screwdriver and rescrew it onto the clip arm at one of 5 possible positions.

The clip should generally lie in the lower line of sight through the eyeglasses so that you can still orient yourself in the room by looking through the upper part of the eyeglasses and over the clip.

If you cannot achieve good results with the “Max-Detail Clip”, this may be a result of one of the following situations:

- You may have selected a working distance that is too far or too short.
- You may be experiencing a vertical offset because you have attached the “MaxDetail Clip” at too steep an angle.
- Your pupil distance may lie outside of the pupil distance range of the “MaxDetail Clip”.

Care Instructions

Your “MaxDetail Clip” is a technically and optically advanced vision product and requires appropriate handling and care. We recommend using the provided case for storage to protect the lenses from dust and external influences.

- **Do not use alcohol or organic solvents to clean the “MaxDetail Clip”!**
- **Do not clean the “MaxDetail Clip” in an ultrasound bath!**

Clean the “MaxDetail Clip” with a soft cotton or linen cloth (e.g. lens cleaning cloth). In case of heavy soiling (e.g. fingerprints) moisten the cleaning cloth slightly.

Technical Data

- ▶ Working distance: 350 mm from lens crown
approx. 400 mm from eye
- ▶ Pupil distance range: 62 - 70 mm
- ▶ Magnification: 2x
- ▶ Field of vision: approx. 85 mm
- ▶ Weight: 23 g

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Eschenbach Optik GmbH