# maxTV<sup>®</sup> Clip

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**User Manual**

**Mode d'emploi**

**Istruzioni d'uso Manual**

**de instrucciones**

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**Bruksanvisning**

**Bruksanvisning**

**Käyttöopas**

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**ESCHENBACH**

Deutsch .....	3
English.....	7
Français.....	10
Italiano .....	14
Español.....	17
Nederlands.....	21
Dansk.....	24
Svenska.....	27
Norsk.....	31
Suomi.....	34
Polski.....	37
Česky .....	41
日本語.....	44

## Deutsch

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## English

You have purchased a quality product from the Eschenbach company that was 'Made in Germany' using state-of-the-art manufacturing processes. We congratulate you on your decision.

MaxTV Clip is a magnifying attachment for clipping onto existing eyeglasses. This allows you to see twice as large at a distance of about 3 m without straining your eyes. That makes it ideal use for watching television. Modern design and easy use were the main priorities during development of this product.

### Safety Information

- ▶ Danger of glare and injury! Never look directly into the sun or other bright light sources using optical devices!
- ▶ Danger of fire!  
Lenses in optical devices can cause significant damage by focusing light to generate heat if handled or stored improperly. Be careful never to leave optical lenses lying in direct sunlight!
- ▶ Also inform other people and children especially of this safety measure!

- ▶ MaxTV Clip may not be used while driving automobiles.
- ▶ Protect your 'MaxTV Clip' from impacts and excessive heat. Never lay your 'MaxTV Clip' on a radiator or in direct sunlight.
- ▶ Be particularly careful when using the product on eyeglasses with plastic lenses. When used improperly, the eyeglasses may be scratched since the lens of the product rests directly against the lens of the eyeglasses.

## **Adjustment**

The lens of the 'MaxTV Clip' is designed for a working distance of about 3 m.

To attach the 'MaxTV Clip' to your eyeglasses, press the clip together.

When you wish to stop using the 'MaxTV Clip', simply flip the clip up, it is not necessary to remove it from the glasses.

To allow an optimal line of sight on both large and small eyeglasses, the lens assembly on the clip can be moved vertically. To do this, it is necessary to unscrew the lens assembly with a small Philips screwdriver and rescrew it onto the clip arm at one of 5 possible positions.

If you cannot achieve good results with the ‘Max-TV Clip’, this may be a result of one of the following situations:

- You may have selected a working distance that is too far or too short.
- You may be experiencing a vertical offset because you have attached the ‘MaxTV Clip’ at too steep an angle.
- Your pupil distance may lie outside of the pupil distance range of the ‘MaxTV Clip’.

## Care Instructions

Your ‘MaxTV Clip’ is a technically and optically advanced vision product and requires appropriate handling and care. We recommend using the provided case for storage to protect the lenses from dust and external influences.

- **Do not use alcohol or organic solvents to clean the ‘MaxTV Clip’!**
- **Do not clean the “MaxTV Clip” in an ultrasound bath!**

Clean the ‘MaxTV Clip’ with a soft cotton or linen cloth (e.g. lens cleaning cloth). In case of heavy soiling (e.g. fingerprints) moisten the cleaning cloth slightly.

## Technical Data

- ▶ Working distance: approx. 3 m
- ▶ Pupil distance range: 60 - 68 mm
- ▶ Magnification: 2 ×
- ▶ Field of vision: 9° = 1 m / 3 m
- ▶ Weight: 28.6 g

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Eschenbach Optik GmbH