# menas LUX

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User Manual

Mode d'emploi

Istruzioni d'uso

Manual de instrucciones

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Bruksanvisning

Käyttöopas

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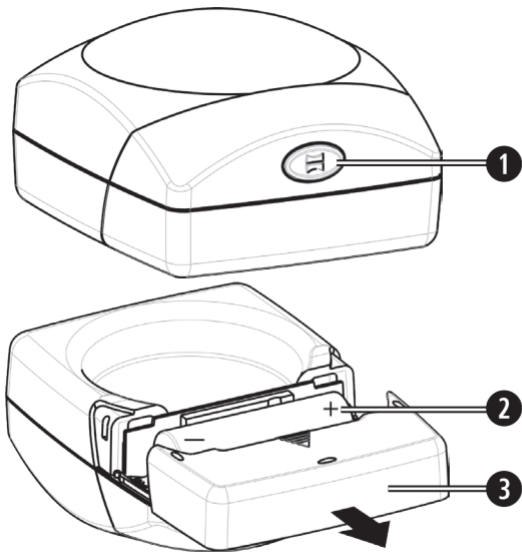
 **ESCHENBACH**

Deutsch .....	4
English.....	9
Français.....	14
Italiano.....	19
Español.....	24
Nederlands.....	29
Dansk .....	34
Svenska.....	38
Norsk.....	42
Suomi.....	47
Polski.....	52
Česky .....	57
日本語.....	62

## Deutsch

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## English

You have purchased a quality product made with the latest production technology by ESCHENBACH in Germany; a brand product „made in Germany“. We would like to congratulate you on this decision.

### Safety information

▶ **Danger of fire!**

**Lenses in optical devices can cause significant damage by focusing light to generate heat if handled or stored improperly. Be careful never to leave optical lenses lying in direct sunlight!**

▶ **Risk of blinding and injury!**

**Never look directly into the sun or other bright light sources using optical devices!**

- ▶ **Protect your menasLUX from impacts and excessive heat. Never lay your menasLUX on a radiator or in direct sunlight.**
- ▶ **Also inform other people and children especially of this safety measure!**

## **Using the menasLUX**

menasLUX is handy magnifier placed directly on the text to be read so that is presented larger.

1. Place the menasLUX on the text that is to be magnified.
2. Press the button **①**, to switch the light on.
3. Move the menasLUX over the text to read it.
4. After use, switch the light off by pressing the **①** button once again.

## **Changing batteries**

menasLUX is powered by a 1.5 V battery (Size AA, Mignon) for the LED light. Should the brightness of the light diminish,

it means the battery is discharged and must be replaced.  
Proceed as follows.

1. Switch the light off.
2. Open the battery compartment ❸, by removing the battery cover on the underside of housing in the direction of the arrow.
3. Replace the battery ❷ with a new one of the same type. Pay attention to the correct polarity.
4. Close the battery compartment by sliding the battery cover in the opposite direction of the arrow on the housing.

## **Technical Data**

- System Magnification: 3.0x, 12 diopters
- Battery: 1.5V-Battery, Size AA, Mignon

## **Disposal**

When the device reaches the end of its useful life, do not dispose of it with normal household waste. This product is subject to European directive 2002/96/EC.

Dispose of the product or parts of the product through a certified disposal agency or your municipal disposal agency. Observe the currently applicable regulations. In case of doubt, contact your disposal agency. Dispose of all packaging materials in an environmentally friendly manner. Batteries and cells must not be disposed of in household waste! Provide for proper disposal.

## **Instructions on Care and Maintenance**

Clean the menasLUX with a soft, lint-free cloth (i.e. eye glasses polishing cloth). With heavy soiling (i.e. fingerprints) clean the menasLUX carefully with a moist cloth. Never use alcoholic and/or organic solvents to clean the menasLUX as these can destroy it.

## **Warranty**

Within the parameters of the applicable statutory provisions, we provide a warranty for the functioning of the product described in this manual with regard to faults arising in connection with manufacturing or material defects. In the event of damage due to improper handling, including damage from a fall or impact, no warranty claim exists. Warranty claims are only accepted upon presentation of the sale receipt!

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Eschenbach Optik GmbH