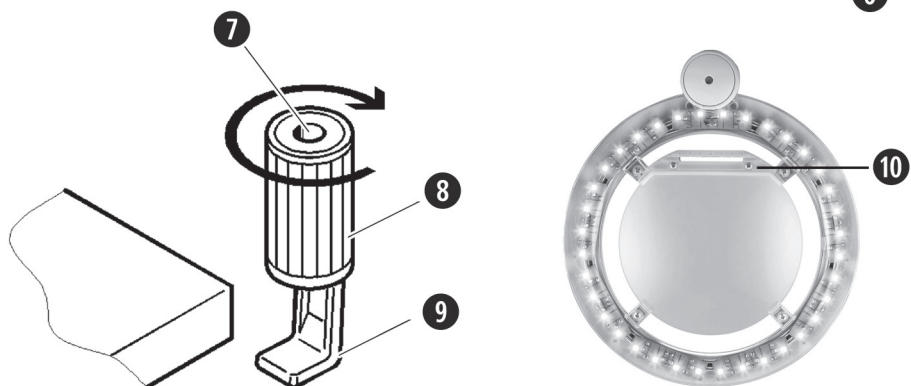
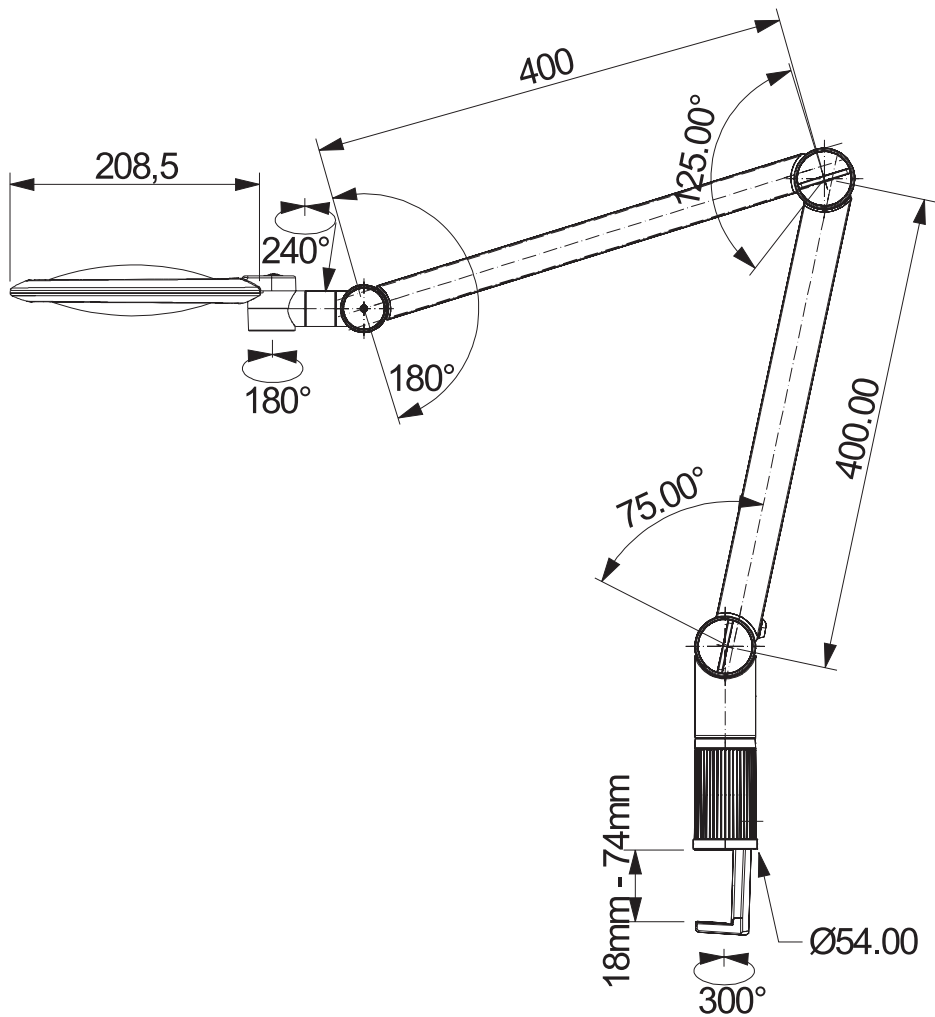
varioLED

Bedienungsanleitung
User Manual
Mode d'emploi
Istruzioni d'uso
Manual de instrucciones
Gebruiksaanwijzing
Betjeningsvejledning
Bruksanvisning
Käyttöopas
Instrukcja obsługi
Provozní návod
サービスマニュアル

Deutsch	4
English.....	6
Français.....	8
Italiano.....	10
Español.....	12
Nederlands.....	14
Dansk	16

Svenska.....	18
Norsk.....	20
Suomi.....	21
Polski.....	24
Česky	26
日本語.....	29

Deutsch

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English

You have purchased a quality product from the Eschenbach company that was "Made in Germany" using state-of-the-art manufacturing processes. We congratulate you on your decision.

The lens of high-quality, unbreakable PXM® material is protected with a cera-tec® hard coating. Scratch-resistance is an important aspect of the quality and useful life of the lens.

Safety Instructions

- ▶ **Danger of glare and injury!**
Never look directly into the sun or other bright light sources using optical devices!
- ▶ **Risk Group 1 as per EN 62471:2008.**
Risk value of the exposure (EHV): 2200 seconds in 800 mm.
The limit value of the Free Group is maintained from 1.8 m spacing.
Retinal risks arising from blue light 400 nm up to 780 nm.
Do not look into the light source for a long period.

- ▶ **Danger of fire!** Lenses in optical devices can cause significant damage by focusing light to generate heat if handled or stored improperly. Be careful never to leave optical lenses lying in direct sunlight!
- ▶ Protect your magnifying light from impacts, moisture and excess heat. Never lay your magnifying light on a radiator or in direct sunlight.
- ▶ The clamping screws ④ must be screwed in tight during use of the magnifying light!
- ▶ Never use the magnifying light near to containers filled with water!
- ▶ Risk of tripping! Make certain that the power cord does not pose a danger. Do not wind the power cord around the magnifying light.
- ▶ Inform other persons, particularly children, of these dangers.

The Operating Elements

- ① Lens
- ② Protective lid
- ③ On/Off switch
- ④ Clamping screws
- ⑤ Pivot arm
- ⑥ Multi-range AC adapter with power cord (110-240V~/50-60Hz)
- ⑦ Hole for pivot arm
- ⑧ Clamping cylinder
- ⑨ Table clamp
- ⑩ Lens holder

Setting Up

1. Screw the table clamp ⑨ to a table surface with the clamping cylinder ⑧.
2. Insert the pivot arm ⑤ into the hole ⑦ of the table clamp ⑨.
3. Plug the AC adapter ⑥ into a properly connected power socket. When using the device outside of the EU, please use the country-specific adapter piece.

4. Open the protective lid ②. If necessary, you can also remove this entirely. The lid can be snapped in or out of the pivot axis at any position.
5. Turn the control knob ③ back past the stiff point to switch on the magnifying light. A slight warming of the outer light ring during operation is normal. Under unfavourable ambient conditions, the warming of the device may increase. If the temperature reaches 65°C, the output is automatically reduced.
6. Turn the control knob ③ farther back to increase the brightness. Turn it forward to reduce the brightness.
7. Position the magnifying light at the desired working height.
8. To switch off the device, turn the control knob ③ all the way forward past the stiff point.
9. To disconnect the device from the power supply, unplug the AC adapter ⑥ from the socket.
10. After use, close the protective lid ② to protect the lens ① from dust and soiling.

Accessories

We offer a lens with 6.0 diopters as an accessory for Art.-No. 27 770. (magnification 2.5x, diameter 132 mm, article no.: 277701).

- ▶ The lens is changed via the rear lens holder ⑩ (in-cut part of the lens ①). Turn the magnifying glass head to a vertical position so that the lens ① does not fall out of the holder when loosening the two screws. Loosen the two screws, then remove the holder ⑩. Take the lens ① out of the holder and insert the 6 dpt lens into the holder. Make certain that it is positioned correctly! The larger curvature must face down.

A transparent lens protection cover for clamping under the lens ① is also available

for work that produces dust or chips (article no.: 277702).

Care Instructions

When the magnifying light is not in use, it should be covered with the protective lid ②.

Pull the AC adapter out of the socket before cleaning!

Clean the housing and arm with a damp cloth. Clean the lens ① with a soft, lint-free cloth (e.g. glasses cleaning cloth). In case of heavy soiling (e.g. fingerprints), clean the lens ① carefully with a damp cloth.

Do not use any soap solutions that contain softening agents, alcohol-based solvents or abrasive cleansers! These could destroy the lens ①.

Technical Data

- ▶ Optics:

27770	27771
• Refractive power: 3.55 dpt	6,0 dpt
• Magnification: 1.9x	2,5x
• Diameter: 132 mm	132mm
- ▶ Illumination:
 - Infinitely adjustable, from 50% - 100%
 - Illumination strength:
 - max: approx. 5000 lx at 200 mm,
 - min: approx. 2000 lx at 200 mm
- ▶ LED class 1 EN 60825-1:2002

Warranty

Within the framework of the applicable statutory provisions, we provide a warranty for the functioning of the product described in this manual with regard to faults arising in connection with manufacturing errors or material errors. In event of damage due to improper handling, including damage from a fall or impact, no warranty claim exists. Warranty claims are only accepted upon presentation of the sale receipt!

Disposal

When the device reaches the end of its useful life, do not dispose of it with normal household waste.

This product is subject to European directive 2002/96/EC.

Dispose of the product or parts of the product through a certified disposal agency or your municipal disposal agency. Observe the currently applicable regulations. In case of doubt, contact your disposal agency.

Dispose of all packaging materials in an environmentally friendly manner.

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Eschenbach Optik GmbH