varioLED

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Lens Changing Instructions

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- ▶ Brechkraft: 6,0 dpt
- ▶ Vergrößerung: 2,5x
- ▶ Durchmesser: 132mm

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English

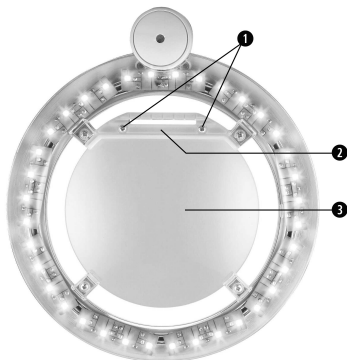
Safety Instructions

- ▶ Danger of glare and injury! Never look directly into the sun or other bright light sources using optical devices! Never look directly into the light source.
- ▶ Danger of fire! Lenses can cause significant damage by focusing light to generate heat if handled or stored improperly. Be careful never to leave optical lenses lying in direct sunlight!
- ▶ Protect the replacement lens from impacts, moisture and excess heat.
- ▶ Inform other persons, particularly children, of these dangers.

Lens Change

The lens is changed via the rear lens holder ②. Turn the magnifying glass head to a vertical position so that the lens ③ does not fall out of the holder when loosening the two screws ①. Loosen the two screws ①, then remove the holder ②. Take the lens ③ out of the holder and insert the 6-dpt lens into the holder. Make certain that it is positioned correctly! The larger curvature must face down.

Care Instructions

Clean the lens ③ with a soft, lint-free cloth (e.g. glasses cleaning cloth). In case of heavy soiling (e.g. fingerprints), clean the lens ③ carefully with a damp cloth.

Do not use any soap solutions that contain softening agents, alcohol-based solvents or abrasive cleansers! These could destroy the lens ③.

Technical Data

- ▶ Refractive power: 6.0 dpt
- ▶ Magnification: 2.5x
- ▶ Diameter: 132 mm

Warranty

Within the framework of the applicable statutory provisions, we provide a warranty for the functioning of the product described in this manual with regard to faults arising in connection with manufacturing errors or material errors. In event of damage due to improper handling, including damage from a fall or impact, no warranty claim exists. Warranty claims are only accepted upon presentation of the sale receipt!

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Eschenbach Optik GmbH