
visomax

Bedienungsanleitung
User Manual
Mode d'emploi
Istruzioni d'uso
Manual de instrucciones
Gebruiksaanwijzing
Betjeningsvejledning
Bruksanvisning
Käyttöopas
Instrukcja obsługi
Provozní návod
サービスマニュアル


ESCHENBACH

Deutsch	5
English.....	5
Français	6
Italiano.....	7
Español.....	7
Nederlands	8
Dansk	9
Svenska.....	9
Norsk.....	10
Suomi.....	11
Polski.....	11
Česki	12
日本語.....	13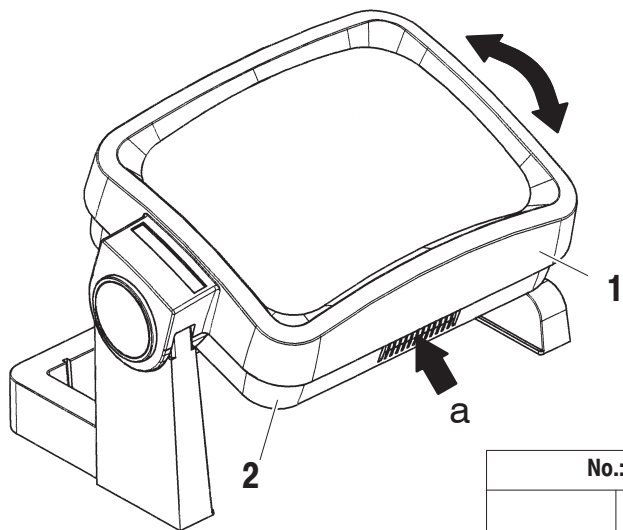

No.: 263611	
	
1 ⇒	7,0 Dpt = 2,8 x
1 + 2 ⇒	11,0 Dpt = 3,8 x

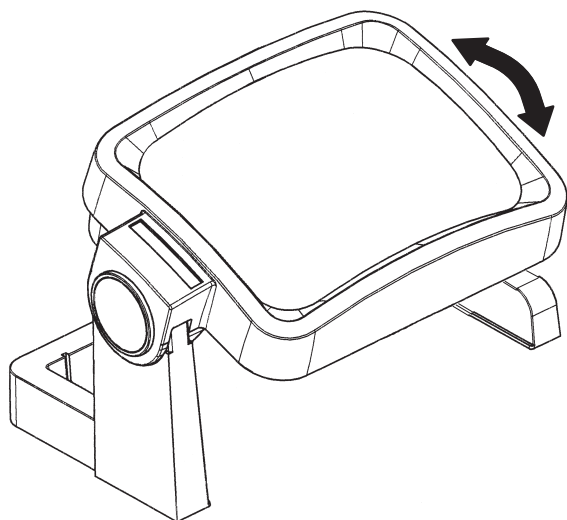No.: 26367	
	
7,0 Dpt = 2,8 x	

Deutsch

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English

Free-standing reading lenses by Eschenbach. For better quality of life.

You have selected a quality product from the house of Eschenbach which has been produced using the latest manufacturing processes in Germany, a top-class "Made in Germany" product. We congratulate you on your choice.

Safety instructions

- **Danger of fire!**
Lenses used in optical equipment may cause considerable damage through their "burning glass effect" if used or stored incorrectly. Ensure that optical lenses are never left uncovered in the sunlight!
- **Danger of blinding or injury**
Never look at the sun through any optical equipment!
- **Protect your free-standing reading lens against jolts and impact shocks, moisture and excessive heat. Never place your free-standing reading lens on heaters or in the sunlight!**
- **Ensure that other people, especially children, are aware of these dangers!**

Operating instructions

- [1] Main lens
- [2] Additional lens (removable),
item no. 263611
1. Place the free-standing reading lens on the text you want to read.
 2. Tilt the lens to suit your angle of view.

Remove additional lens

1. Place free-standing reading lens on a base.
 2. To unhitch the additional lens [2], press in the direction of arrow [a] and remove the additional lens by pulling downwards.
- For reassembly, the additional lens engages under the main lens [1].

Instructions for care

When the free-standing reading lens is not in use, it should be kept covered.

Clean the housing with a damp cloth.

Clean the lenses using a soft, lint-free cloth (e. g. spectacle cloth).

Where the lens has become very dirty (e.g. fingerprints), clean the lenses carefully with lukewarm water and take care while drying them off.

Do not use any soap solutions that contain softeners, alcohol-based solvents or abrasive cleaning products. These may damage the lenses.

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Guarantee

We are required by law to guarantee the function of the product described in these descriptions in respect of defects caused by manufacturing or material faults. Any damages caused by incorrect handling, including damage caused by being dropped or knocked, are not covered under the guarantee. Guarantee benefits can only be claimed through proof of purchase in the form of a sales receipt!

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Eschenbach Optik GmbH