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DE

Operating Instructions

EN

Mode d'emploi

FR

Istruzioni d'uso

IT

Manual de instrucciones

ES

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NL

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DA

Bruksanvisning

SV

Bruksanvisning

NO

Käyttöohje

FI

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PL

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CZ

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JP

optaro[®] XL

Deutsch

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English

About these instructions	24
Safety.....	24
Technical specifications	26
Product description	27
Power supply	29
Download the app	30
Switching on and off.....	30
Layout of the device	30
Commissioning	30
optaro XL app	32
Troubleshooting	42
Care instructions	42
Firmware update	42
Disposal	43
Applicable standards.....	43
Warranty	45
Trademarks.....	45

About these instructions

These instructions contain important information for the safe use and proper cleaning of your product.

Read these instructions and the safety instructions (you will find these in a separate booklet in the packaging) carefully and thoroughly before using this product.

Keep these instructions for future reference.

These instructions are based on the app version "1.0.0" or higher.

Safety

Intended use

This electronic reader is intended exclusively for enlarging and reading or viewing texts or images. To allow for this function, the reader must be connected with an iPad (with USB-C charge socket).

This product is intended for use in a domestic environment only (including care homes, workplaces, schools and other educational establishments).

Safety instructions

Important information for your safety

Read and follow the following safety instructions carefully!

Make other people - especially children - aware of these dangers.

- Risk of glare and injury! Do not look into the light sources on the underside of the housing.
- Do not allow children to handle electrical appliances unsupervised. The product is not a toy!
- Choking hazard and danger of swallowing: Keep packaging, foils and packaging parts away from children.
- Protect the device from impact or shock. Handle the device with care.
- Only set up the device on a suitable surface, e.g. a table
- Lay the USB cable so that nobody can trip over it.
- Check the device and the cable for visible damage. In the event of damage, disconnect the connected mains adapter from the power supply immediately. Have the product checked by qualified specialists before using it again.
- Risk of trapping: Be careful not to trap your fingers when folding the device.
- Never open the device housing. Opening it will invalidate the warranty.
- Do not use the device while walking.

- We shall not be liable for the correctness of the spoken contents of the OCR function. This is particularly important when processing sensitive information such as the dosage of medication.
- Pay attention to your iPad! The iPad is only placed loosely on the reader, but is not physically connected with it. When the reader is moved, in particular if the support is folded up, the iPad can fall down and be damaged.

Correct handling of rechargeable batteries

- The built-in battery may only be removed or replaced by authorised specialist personnel.
- Only use approved battery models! Only use the battery specified and released by us for this product. This battery was comprehensively tested with regard to safety, performance and compatibility. Using any other battery than the one mentioned will void the warranty and any product liability claims. In that case, we shall not assume any responsibility for damage, failures or consequential damage.
- Safety risks with third-party batteries! The use of non-approved batteries can lead to overheating, leakage, fire or an explosion risk. In addition, the function of integrated protection circuits can be impaired.
- Dispose of used batteries in accordance with the legal requirements.
- Never throw a rechargeable battery into a fire.
- Never crush or cut a rechargeable battery.
- Never leave a battery in an environment with extremely high temperatures, as this can lead to an explosion or the release of flammable liquids or gases.
- Never expose a battery to extremely low air pressure, as this can lead to an explosion or the release of flammable liquids or gases.

Notes on the usage environment

- Do not charge the device in places with excessive heat. High ambient temperatures can affect the charging process.
- Never expose the device to moisture or wetness. The device is not waterproof!
- Do not store the device in places with excessive heat, such as in a parked car.
- Protect the device from excessive heat and naked flames. Never place the device on radiators or in direct sunlight.
- In a medical environment, this device must not be operated with other life support equipment.

Scope of delivery

- optaro XL
- long USB cable (type C to type C to charge the battery)
- short USB cable (type C to type C to connect with the iPad)
- Operating instructions

Technical specifications

Magnification range	1.0x to 22x (depending on the end device used)
Display modes	True colour 14 contrast-enhanced contrast modes
Reading camera	Initial magnification of 2.5x
Read-aloud camera	Initial magnification of 1.0x
Aspect ratio	4:3
Resolution	2048x1536 or other resolutions in the aspect ratio
Illumination	LED
Recommended power source	USB-PD (5V=3 A/9 V=2.0 A/12 V=1.66 A)
USB connection (charging)	Type C (long cable)
USB connection (connection with iPad)	Type C (short cable)
Conducted power	<5 dBm
Radio network	2402~2480 MHz
Battery	Lithium-polymer
Operating time	approx. 6.5 hours
Charging time	approx. 4 hours
Dimensions (folded, iPads up to 11")	209 mm x 316 mm x 62 mm
Dimensions (folded, iPads up to 13")	224 mm x 320 mm x 60 mm
Weight	1240 g
Operating temperature range	0 °C - 40 °C (32° F - 104° F)

Product description

The optaro XL electronic reader is a medical device for people with impaired vision and is used to magnify reading material (e.g. text and images).

Display position 1

Display position 2

The reader is powered by its own battery. The battery is charged via USB-C and has an operating time of approx. 6.5 hours.

The foldable stand holds the reader in position.

The product is controlled via the “optaro XL app”.


The reading material is recorded via the reader’s cameras. The app magnifies the recorded data on the display of the iPad. The reader enables 1x to 22x magnification (depending on the mobile device), various false colour modes and is equipped with two fixed-focus cameras.

Other functions include the read aloud function, the photo function, the display of a reading aid and an overview display.

Compatible iPads

optaro XL can be used with all iPads (iPadOS 18.0 or higher) with USB-C charge socket.

Operating and functional elements

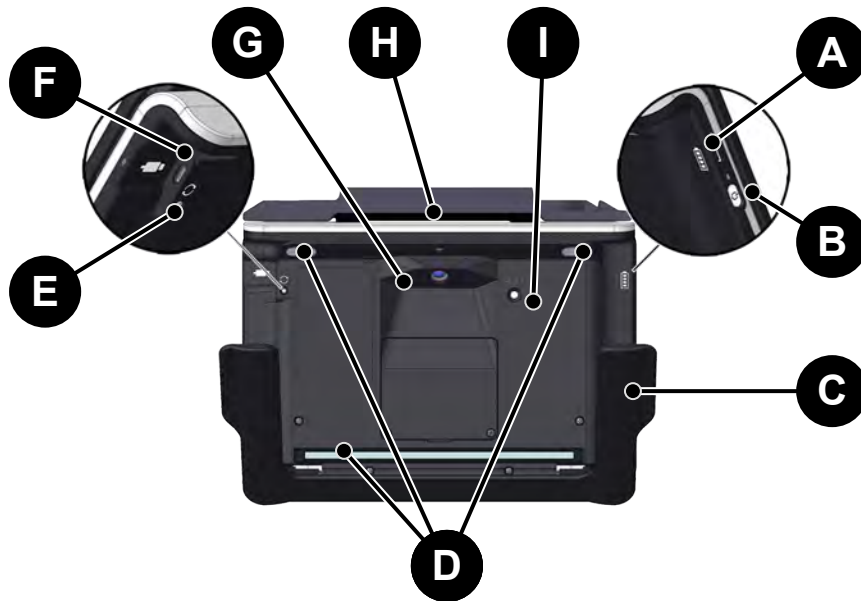- A USB-C charging port / LED charging indicator
- B On / Off button / LED - operating indicator
- C Foldable stand
- D LED illumination
- E Update button
- F USB-C connection to iPad / LED - connection indicator
- G 2 cameras
- H iPad layer
- I RESET button

LED indicators and explanations

The device is equipped with three LEDs that indicate different operating states:

LED charging indicator (A)

Status	LED colour
Device is charging	Red
Battery fully charged (100%)	Green

LED operating indicator (B)

Status	LED colour
Device is switching on	Blue (flashing)
Device is switched on	Blue (permanent)

LED connection indicator (UVC) (F)

Status	LED colour
Connection between optaro XL and iPad active	Green
No connection (e.g. no cable connection or app closed/minimised)	Off

Power supply

For optimum operation, a power supply with a minimum output of 20 Watt is recommended. Please ensure that the power supply used meets these requirements.

Note regarding the charging process:

During charging, optaro XL always takes precedence over the iPad. If the power supply does not supply the required output of at least 20 Watt, the iPad will possibly not be charged.

Download the app

The optaro XL app is freely available in the Apple App Store and can be downloaded there.

The app can also be downloaded using the QR code located on the back.

Download the app and start the app. Follow the instructions on the screen for initial operation.

Switching on and off

To switch the device on and off, press and hold the On/Off button (B) for one second.

Automatic switch-off

The camera module switches off automatically after 20 minutes if the “optaro XL app” is not connected.

Layout of the device

Risk of trapping

Mind your fingers during folding and unfolding.

Commissioning

Preparation:

- Compatible iPad
To operate the optaro XL reader, you require a compatible iPad (not included in the scope of delivery).
- Short USB-C cable for the connection
- optaro XL app downloaded and set up on the iPad

To set up the optaro reader XL, proceed as follows:

1. Unfold the stand (C) and place the iPad into the desired display position (see page 27).

2. Place the iPad into the support.

- Pay attention that the On/Off switch and the volume buttons of the iPad rest in the recess. An iPad camera is outlined on the support (H) to assist with the correct placement of the iPad.

3. Take the short USB-C cable and insert one of its connectors into the USB-C socket (F) of optaro XL (on the right hand side of the device).

- The cable can remain permanently plugged in there even if you remove the iPad.


4. Remove the other end of the USB-C cable and insert it in the USB-C socket of the iPad.
5. Switch on the reader by pressing the On/Off button (B).
- The status LED above lights up blue when the camera module is ready.
6. Ensure that the Bluetooth connection between optaro XL and iPad has been established.
7. Start the optaro XL app on your iPad.
- ✓ The reader is ready for use.

optaro XL app

Display screen

The main screen of the optaro XL app is structured as follows and offers various functions: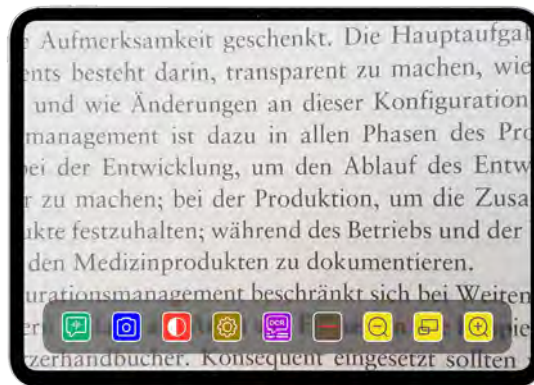


Icon	Name	Meaning
	"Tell me" function	Starts the read aloud function for the visible reading area
	Take a photo	Take a photo and switch to photo mode
	Activate/deactivate false colours	Switch between the selected false colour
	Settings	Opens the "Settings" menu
	Read aloud function	Change to OCR function (optical character recognition).
	Activate/deactivate reading support	Activates/deactivates reading support
	Zoom out	Reducing the zoom factor
	Overview function	Switches between the smallest and the previously set magnification
	Zoom in	Increasing the zoom factor

Functions

“Tell me” function

You can use the “Tell me” function to have the visible reading area read aloud. The area read aloud is highlighted with a green border.

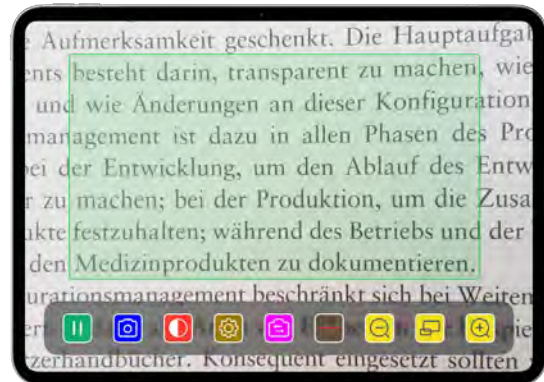Important information on voice output


The function is not available in all languages.¹

Take a photo

In photo mode (marked by the blue frame at the edge of the screen), you can save the current screen section to your gallery, open your gallery or share the displayed screen section.

optaro XL saves the photos in a separate album named “optaro XL” (iOS requires permission to access the photo library).

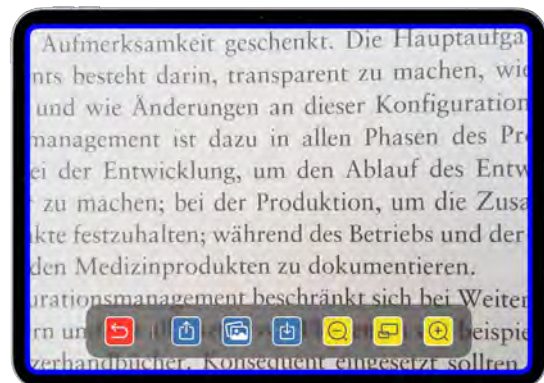	Save	Saves the displayed screen section as a photo in the gallery.
	Photo gallery	Opens the “Photo gallery” of the iPad
	Share photos	Opens the menu for sharing and sending the photo
	Back button	Exits the photo mode and switches back to the live mage

¹ Available languages are: German, English, French, Italian and Spanish

False colours


To display the camera image in false colours, switch between the various false-colour modes or return to the true colour display, press repeatedly

Settings

Press  to call up the settings menu. The following settings can be made: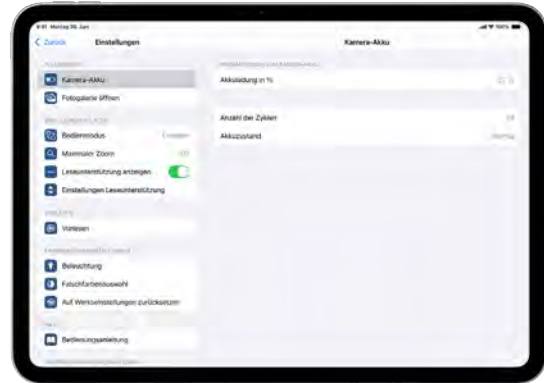

Camera battery

You can view the current battery status here.

Open photo gallery



You can access your albums in the photo gallery. The 'optaro XL' album shows all photos taken with optaro XL. The photos can be enlarged or reduced accordingly. You also have the option of sending the photos (see chapter "Take a photo" on page 33).


Operating mode

Here you have the option of changing the operating mode. The 'Advanced' mode is displayed by default.


The active selection is marked with a check mark. The operating mode can be set by pressing the respective button.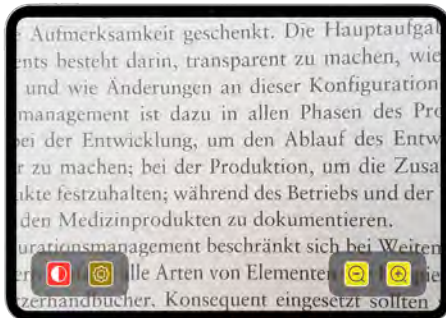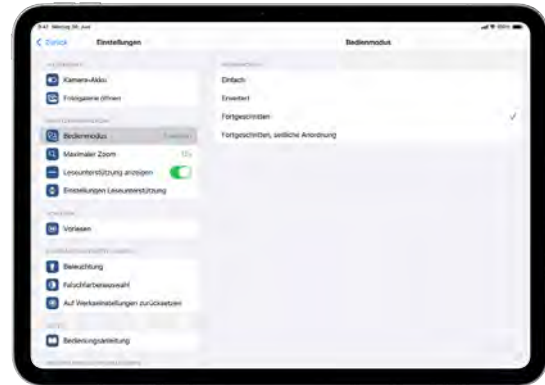


Operating mode 'Simple'

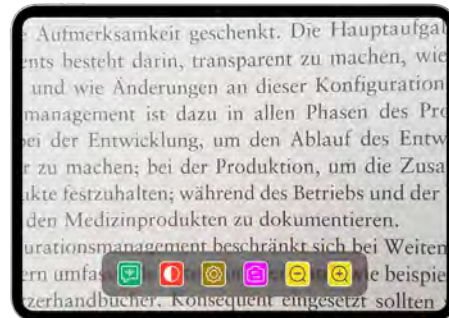Operating mode 'Enhanced'

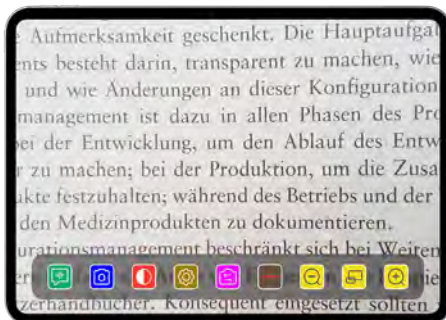Operating mode 'Advanced'

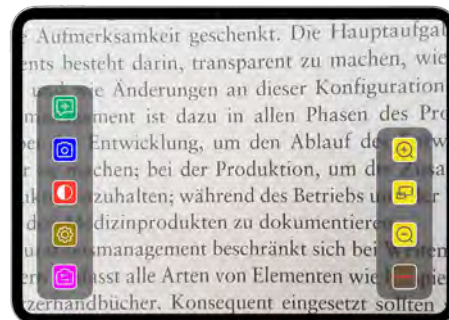Operating mode 'Advanced, lateral arrangement'

Maximum zoom

Here you can set the maximum magnification using the slider. A maximum magnification of 15x is displayed by default.

Show/hide reading support

Reading support (reading line / reading bar) can be activated or deactivated here. In advanced operating mode, the reading support can also be displayed or hidden via .

Reading support settings

Here you have the option of adapting the reading support.


	Move reading line upwards	You can use this button to move the reading line upwards. Alternatively, you can also move the reading line by sliding it with your finger.
	Move reading line downwards	You can use this button to move the reading line downwards. Alternatively, you can also move the reading line by sliding it with your finger.
	Switch between reading line and reading bar	In addition to the reading line, a reading bar can also be set. This reading line can be adjusted in height by pressing the  and  .
	Vertical/horizontal reading support	Switching from horizontal reading line / reading bar to vertical reading line / reading bar can be set here.
	Line thickness	The thickness of the reading line can be adjusted by pressing this button. Three different thicknesses can be selected.
	Colour of the reading line	The colour of the reading line can be adjusted by pressing this button. In addition to the red reading line (set as standard), a black or grey reading line can also be used.

Read aloud function

In this menu, different settings can be made for the OCR function.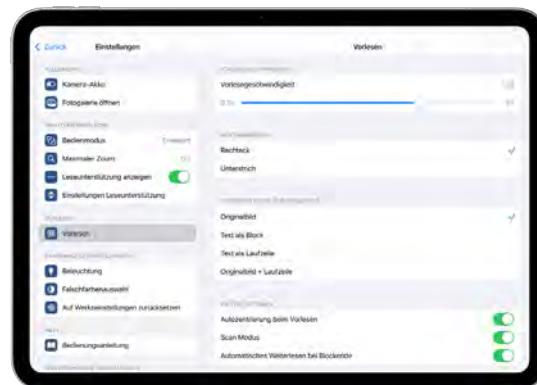

Reading speed


Here you can adjust the reading speed using the slider. Single speed is used by default.

Marking of words


Here you can select how read words are marked in the read aloud function. The active selection is marked with a check mark.

Supporting read aloud view


Here you can adjust how the read aloud view is displayed. 4 modes are available for selection. The active selection is marked with a check mark.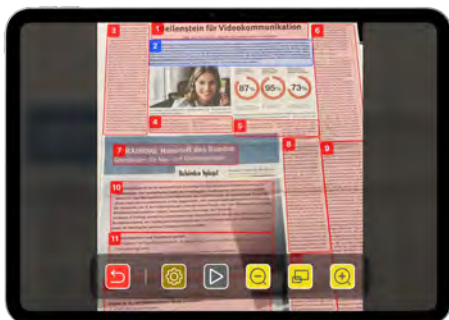

Original image

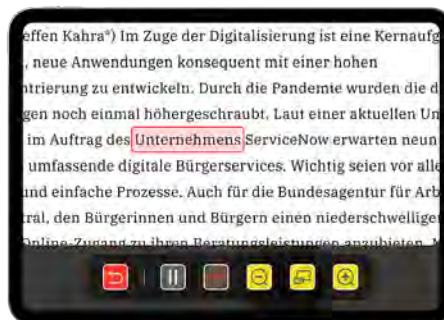Text as block

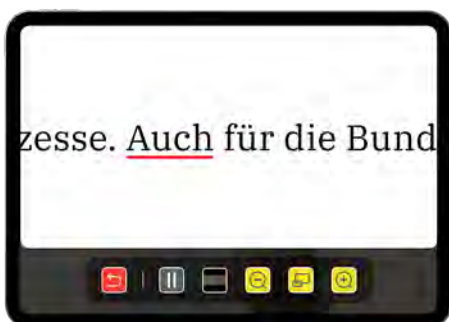Text as scrolling line

Original image + scrolling line

Auto centring

In auto centring, the read word is followed. Move the controller to deactivate auto centring.

OCR scan mode

Switch to activate / deactivate the OCR scan mode (see page 39).

Automatically reading on at end of block

Switch to activate / deactivate automatically reading on at end of block.

Font

Here you can select different fonts for the display of the read aloud function. The active selection is marked with a check mark.

Illumination

In this menu you can adjust the LED illumination of the reading camera.

There are three buttons each for the **top LED** and **bottom LED** function.

The medium illumination is preset by default. Increasing the illumination level will result in a shorter battery life.

	Illumination 50%	Setting the illumination to 50%
	Illumination 100%	Setting the illumination to 100%
	Illumination off	Setting the illumination to 0%

False colours

To select false colour combinations, press this button. The default setting is black/white and white/black. The set false colour combinations are displayed with a tick. A total of 14 different false colour combinations can be set.

Reset to factory settings

Press this button to reset the app to the factory settings.

Operating instructions

The operating instructions can also be viewed on the iPad. To do this, press this button.

About

Displays further information about the manufacturer and the app.

Privacy policy

The privacy policy can also be viewed on the iPad. To do this, press this button.

Terms of use

The terms of use can also be viewed on the iPad. To do this, press this button.

Licence information

The licence information can also be viewed on the iPad. To do this, press this button.

Read aloud function

The “Read aloud function” analyses the current screen section or a larger document. Following that, the device can read the texts out aloud.

	Back button	Exits the read aloud mode and switches back to the live image
	Settings	Opens the “Settings” menu
	Optical character recognition screen	Starts the optical character recognition of the current display.
	Scan a document	Starts the scanning function for documents. Follow the instructions on the screen. The optical character recognition is subsequently opened.

Once the text has been recognised via the button “Optical character recognition screen” or the button “Scan document”, text blocks with numbers are displayed.

To have a block read out aloud, press the desired text block twice.

The read aloud function reads the active text block that is framed in blue.


If the read aloud function is to start reading from a specific word, press the desired word longer.

	Settings	Opens the “Settings” menu
	Play button	Start of the read aloud function (text block 1 is read if no text block is selected)
	Pause button	Pauses the read aloud function
	Zoom out	Reduces the screen section
	Overview function	Switches between the smallest and the previously set magnification
	Zoom in	Enlarges the screen section

Show / hide reading support


You can use the “Show and hide reading support” function to show reading support in the current screen section. You can activate and deactivate the reading support by pressing the  button.

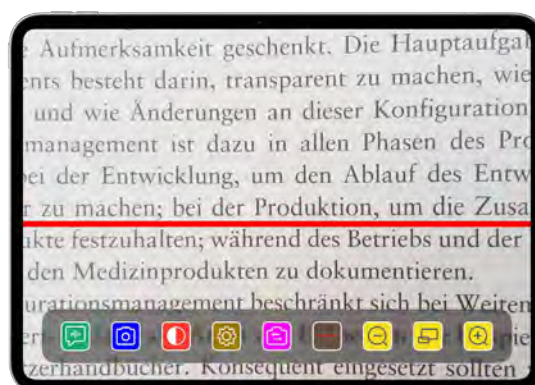Zoom in / Zoom out

You can use the “Zoom in / Zoom out” function to enlarge or reduce the current screen section or switch between the smallest and previously set magnification.

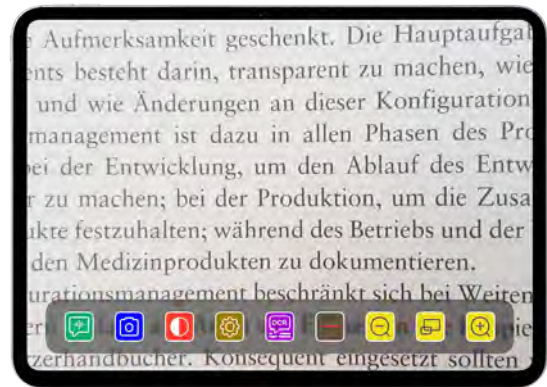	Zoom in	Enlarges the screen section
	Overview function	Switches between the smallest and the previously set magnification
	Zoom out	Reduces the screen section

Troubleshooting

If the device no longer functions as described in these instructions, there may be a fault that you can rectify yourself.

There are several possible solutions that you can try:

1. Check whether the connection cable to the iPad is plugged in.
2. Check whether there is a Bluetooth connection between optaro XL and iPad.
3. Repeat the procedure that did not work.
4. Press the RESET button (H). Then close and restart the app (as described here: <https://support.apple.com/de-de/guide/ipad/ipad79518d15/ipados>). Then repeat the procedure that did not work again.
5. Delete and reinstall the app.
6. In exceptional cases, you may have to switch off your iPad and reinstall the app.

If the above assistance does not solve your problem, please contact your specialist dealer for further support.

Function of the RESET button

If the device does not work as described it may be due to a fault. You can rectify the fault by resetting the device. Press the RESET button for 2 seconds.

Important note: Your personal settings will be retained; there will be no reset to factory settings.

Care instructions

Clean the device with a soft, lint-free cloth. If the optical elements (camera, LEDs) are heavily soiled, e.g. with fingerprints or similar, lightly dampen the cloth.

- Avoid cleaning the device with:
 - Soap solutions that contain plasticisers
 - Alcoholic or organic solvents
 - Abrasive cleaning agents
 - Ultrasonic baths
 - Running water
- Failure to observe the instructions may result in damage to the device.

Firmware update

The current firmware version can be downloaded here:

<https://www.eschenbach-vision.com/de-DE/produkte/produkt/optaror-xl16534>

Disposal

This product is subject to the European WEEE directive on old electrical and electronic equipment in its currently valid version.

Do not dispose of your old device with household waste, but via an authorised waste disposal company or your municipal waste disposal facility.

Observe the regulations applicable in your country. If in doubt, contact your waste disposal organisation. Dispose of all packaging materials in an environmentally friendly manner.

Applicable standards

The device is a Class I medical device (non-sterile, without measuring function) and complies with Regulation 2017/745, Annex VIII, Chapter III, Rule 13. If the device is used outside the EU member states, the respective country-specific directives must be observed.

Hereby, Eschenbach declares that the radio equipment type (art. no. 16534, 165341, 165343) is in compliance with Directive 2014/53/EU.

The applicable declaration of conformity can be found on the following website:

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SL18AA Burnham, Bucks

Internet: www.associatedoptical.com

Hereby, Eschenbach Optik GmbH declares that the radio equipment type optaro XL camera is in compliance with Radio Equipment Regulations 2017. The full text of the UK declaration of conformity is available at the following internet address:

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Supplier's Declaration of Conformity

47 CFR § 2.1077 Compliance Information

Unique Identifier

Product Name: optaro® XL

Model No.: 16534, 165341, 165343

Responsible Party – U.S. Contact Information

Eschenbach Optik of America, Inc.

22 Shelter Rock Lane

Danbury, CT 06810

United States

Tel.: (800) 487-5389

Fax: (888) 799-7200

FCC Compliance Statement

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

HVIN: 16534

MODIFICATION: Any changes or modifications not expressively approved by the grantee of this device could void the user's authority to operate the device.

This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment. End user must follow the specific operating instructions for satisfying RF exposure compliance. This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

IC Warning

ISED Compliance Statements

This device contains licence-exempt transmitter(s)/receiver(s) that comply with Innovation, Science and Economic Development Canada's licence-exempt RSS(s). Operation is subject to the following two conditions:

- (1) This device may not cause interference.
- (2) This device must accept any interference, including interference that may cause undesired operation of the device.

This equipment complies with IC RSS-102 radiation exposure limits set forth for an uncontrolled environment.

RF Exposure Compliance

This equipment complies with ISED radiation exposure limits set forth for an uncontrolled environment. This equipment should be installed and operated with minimum distance 5 mm between the radiator and your body. This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

Warranty

We guarantee the function of the product described in these instructions within the framework of the statutory provisions with regard to any occurring defects that are attributable to manufacturing faults or material defects. Damage caused by improper handling, including damage caused by dropping or impact, is not covered by the warranty. Warranty only by proof of purchase.

Trademarks

USB® is a registered trademark of USB Implementers Forum, Inc.

iPhone, iPad and App Store are trademarks of Apple Inc. registered in the USA and other countries and regions.

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The screenshot shows the 'Einstellungen' (Settings) app on an iPhone. The 'Vorfunktion' (Voice Function) section is selected, displaying various settings for voice control. The 'Vorlesegeschwindigkeit' (Reading Speed) is set to 0.3x. The 'Sprache' (Language) is set to 'Deutsch'. The 'Vorlesegeschwindigkeit' (Reading Speed) is set to 'Originaltext'. The 'Vorlesegeschwindigkeit' (Reading Speed) is set to 'Originaltext'. The 'Vorlesegeschwindigkeit' (Reading Speed) is set to 'Originaltext'.

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