visolux **DIGITAL** XL FHD

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Operating instructions **EN**

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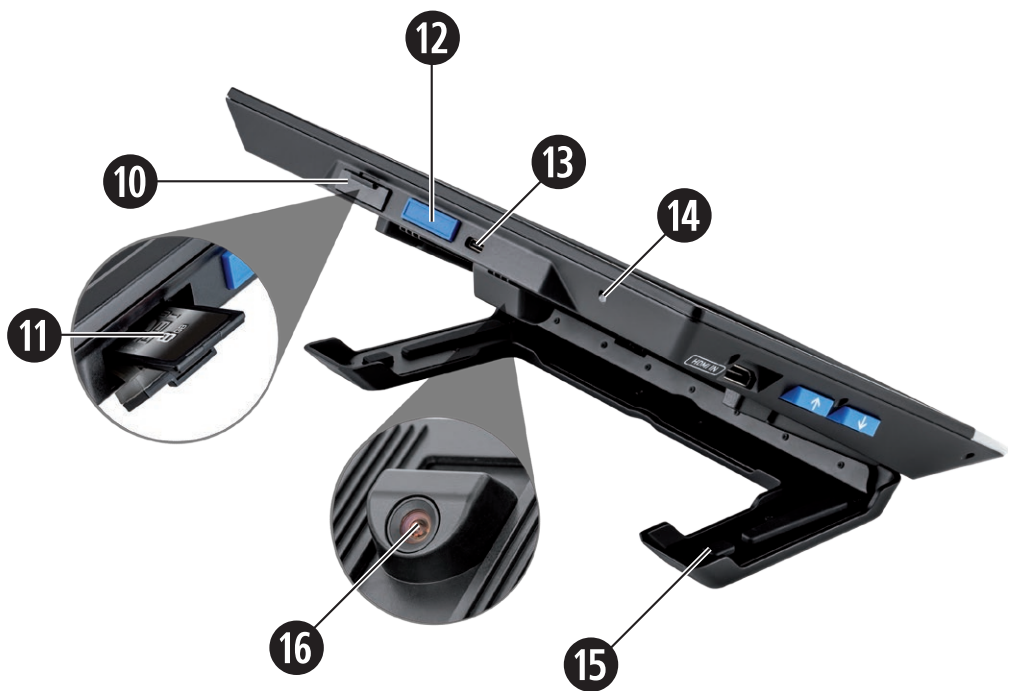
Manual de instrucciones **ES**

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**CE**

**ESCHENBACH**

Deutsch .....	1
English.....	19
Français .....	37
Italiano.....	55
Español.....	73
Nederlands .....	91

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Dear Customer,

We would like to congratulate you on purchasing the Eschenbach visolux DIGITAL XL FHD.

As the market leader for magnifying glasses and reading aids, we at Eschenbach can look back over many years of experience and this is reflected in the quality, usability and design of our products.

## **Firmware version**

This manual is based on firmware version "VD12\_fw\_1.0".

## **Proper use**

This electronic reading device is exclusively for magnifying and reading or looking at texts and images.

## **Before first use**

Please read this manual carefully through before first use. This is the only way to get to know all the functions on the device.

## **Package contents**

- ▶ visolux DIGITAL XL FHD
- ▶ Hard foam case
- ▶ Carry strap
- ▶ Microfibre cloth
- ▶ HDMI cable
- ▶ USB cable (Type C)
- ▶ Universal charging device with adapters for EU, GB, US, AU
- ▶ Operating instructions

## Safety information

- ▶ **Risk of glare and injury!**  
Never look directly at the light source on the underside of the housing!
- ▶ **Do not allow children to handle electronic devices without supervision.**
- ▶ **Only charge the reader with the mains adapter supplied!**
- ▶ **Check the device and the mains adapter with power cable for visible damage. If there is any damage disconnect the connected power unit from the power supply immediately. Ask a qualified specialist to check the product before you use it again.**
- ▶ **Never open the device housing. There are no serviceable parts whatsoever inside. Opening the device will void the guarantee.**
- ▶ **The fitted lithium-polymer batteries may only be removed or exchanged by authorised personnel.**
- ▶ **Never expose the device to moisture.**
- ▶ **When folding the stand feet together make sure you do not catch your fingers.**
- ▶ **Protect the device against impact or knocks and excessive heat.  
Never place the device on radiators or in direct sunlight!**
- ▶ **Ensure that other people, particularly children, are aware of these risks!**

## Operating elements

(Figures see fold-out pages left-to-right)

1. Display
2. Contrast button for calling up contrast functions
3. Photo button for saving images and confirming menu options
4. On/Off Button
5. Button for decreasing zoom factor
6. Button for increasing zoom factor
7. HDMI input
8. Button for moving image selection to right
9. Button for moving image selection to left
10. Cover over SD card slot
11. SD card (insert with contacts in front facing upwards)
12. Button for opening menu control
13. USB / charging connection
14. Charging check LED
15. Foldable stand foot
16. Camera (movable for reading or writing)
17. HDMI output
18. Button for moving image selection upwards
19. Button for moving image selection downwards
20. Reset key for resetting the device (on underside)

## Basic operation

**Before you use the visolux DIGITAL XL FHD for the first time, you must fully charge the internal battery.**

EN

To do this, fit the appropriate country adapter to the power supply. Then plug the power supply into the mains plug and connect the cable to the USB connection ⑬.

The charging indicator lamp illuminates while the device is being supplied with power ⑭.

You can use the device while it is charging and it can remain permanently plugged in.

### Switching on/off

Press the on/off button ④ to switch the device on. After a few seconds the camera image will appear on the display ①. Symbols briefly show the current status, e. g., of battery charging state, zoom factor and display brightness.

Press and hold the On/Off button for ④ for one second to switch the device off. The current settings are saved.

### Reading with the device

Unfold the feet ⑮. Switch the device on and place it on the reading material.

If you are using the device with the minimum zoom factor 2x you will see the complete image section around 12 cm wide on the display. If the zoom is increased, the image section will decrease.

On the rear you can relocate the camera position:

Position at bottom = reading mode, position at top = writing mode

### Dynamic Line Scrolling (DLS)

Thanks to the scrolling function it is not necessary to move the device to one side in order to read a complete newspaper column, for example, even at high zoom factors.

To move the image selection to the right, press the scroll button ⑧ (repeated pressing or keep pressed down).

To move the image selection to the left, press the scroll button ⑨ (repeated pressing or keep pressed down).

Press the scroll button ⑧ twice in quick succession to jump back to the right-hand limit stop. Press the scroll button ⑨ twice in quick succession to jump back to the left-hand limit stop. This enables you to find the beginning of the line quickly.

The scroll facility is also available vertically in the same manner using the scroll buttons ⑱ + ⑲.

### Live touchscreen

You can move the camera image directly on the display. To do this, tap one of your fingers on any point in this display and then swipe your finger across the display.

All scroll functions are only available at zoom factors greater than twofold.

#### **Tip:**

You will find noticeable bulges on the front edge which mark the side-ward limits of the visible area.

The stand foot ⑮ is provided with a notch (see figure) which is arranged in alignment with the camera axis. You can use this notch to place the device over round objects more easily, for example to read information on tins.

### Adapting the zoom

To increase the zoom factor, press and hold the ⊕— button ⑥ when the live image is being displayed. To decrease the zoom factor, press and hold the ⊖— button ⑤. Press the buttons briefly and repeatedly to change the zoom factor in steps.

You can also place two fingers on the display and move them away from each other while still touching the display in order to increase the zoom factor. You can also place two fingers on the display and move them towards each other to decrease the zoom factor.

The zoom factor currently being displayed is displayed briefly top left.

## False colour display

In order to display the camera image in false colours for contrast amplification press the contrast button ②. Press this button again if required in order to switch through the various false colour modes and to return to true colour display. The false colours are preset to black/white and white/black as delivered.

To carry out false colour settings, call up the “Adapting false colours” menu (page 29).

## Saving the live image as a file

Press the photo button briefly ③ to save the current camera image as a file on the inserted SD card and to show it as a still image on the display. You can also change the zoom factor in this image as described above. Press the photo button again to return to the live image display.

## Calling up images from the SD card

To display images previously saved on the SD card, press and hold the photo button ③ for 3 seconds or open the settings menu as described below using the MENU button ⑫ and then tapping the  symbol twice.

You can tap or use the two scroll buttons ⑧ + ⑨ to select the individual symbols which have the functions as described below:

- select the previous image

- select the next image

- select or deselect the current image displayed in the centre as a favourite

- the current image has been set as a favourite

- the current image has not been set as a favourite

**Tip:** The image selected as a favourite is displayed first when the menu is called up. For this reason only *one* image can be selected as a favourite.

- delete the selected image file from the SD card

- back one level

The symbol for the selected function is always shown slightly larger. To activate this function, tap the symbol or press the photo button ③.

To display the image file shown in the centre at full screen size, tap the image. Use the ⊕ and ⊖ ⑥ + ⑤ buttons or the finger movements described above to set the required zoom factor. You can move a zoomed-in image section horizontally across the display using the left-hand scroll buttons ⑧ + ⑨ and you can use the right-hand scroll buttons ⑱ + ⑲ to move it vertically across the display.

**Tip:** You can also relocate the zoomed image using your finger.

If the image is zoomed out you can use the top left-hand scroll button ⑧ or the bottom right-hand scroll button ⑲ to display the next image file at full screen size, and you can use the bottom left-hand scroll button ⑨ or the top right-hand scroll button ⑱ to display the previous image file.

If an image file is being displayed you can press the MENU button ⑫ to display the described 5 menu symbols again.

### *Deleting image files*


In order to delete the image file shown at the centre of the screen from the SD card, tap the delete symbol . If you tap the delete symbol again, the confirmation symbol is shown on the left and the cancel symbol on the right. Tap to delete the file. Tap to cancel the delete process.

Tap twice or press the MENU button ⑫ to return to the live image.

## Settings

Press the MENU button **12** to call up the first level of the menu:

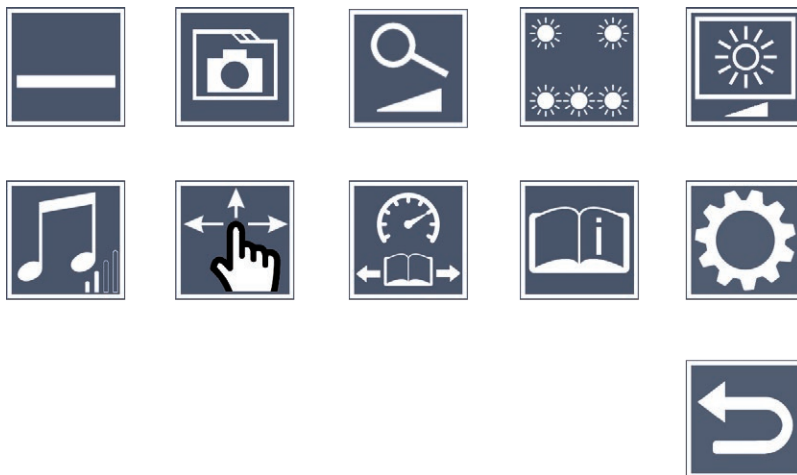EN

You can either move to the individual symbols using the scroll buttons or simply tap on them with your finger. The symbol for the selected function is always shown slightly larger and with a colour marking. To switch the selected function on or off, or to call up a saved submenu, tap the symbol again or press the photo button **3**.

**Tip:** It is preferable to control the functions in the submenus by tapping them on the touch display as well. The buttons described above are also available there.

You can quit the menu at any time by pressing the MENU button **12**.

*Switching reading guideline/line on/off*

Tap once to switch the reading guideline/line on or off. If the reading aid function is switched on, the  symbol is displayed, if the function is switched off the  symbol is displayed.

*Managing photo albums on the SD card*

Tap twice to return to the submenu described in the previous chapter. If you have called up image file management via the settings menu, you can use the  symbol to return to the first level of the settings menu.

### Adjusting the zoom mode

Tap once to switch the infinitely-variable zoom function on or off using the ⊕— ⑤ and ⊖— ⑥ buttons. If the function is switched on, the  symbol is displayed, if the function is switched off the  symbol is displayed.

### Adjusting the reading material lighting

Tap twice to adjust the lighting for the reading material using these symbols:

- switch the complete LED lighting on or off

- only the bottom LED row is switched on (tap twice)

- the top LEDs right and left light up with half power (tap twice)

- the top LEDs right and left light up with full power (tap twice)

If you tap  twice you will return to the first level of the settings menu.

**Tip:** You can also adjust the lighting stages stated above for the top LEDs outside the menu by pressing and holding the contrast button ②.

### Adjusting display brightness

Tap twice and then twice on the symbol for the required display brightness:

- display brightness = 50 %

- display brightness = 75 %

- display brightness = 100 %

### Adjusting the volume

The current volume setting is displayed. Tap repeatedly to select the following settings:

- medium volume

- high volume

- muted

### *Adjusting scrolling*

The symbol for the current scrolling setting is displayed. Tap repeatedly to select the following settings:

- Diagonal image movement possible using live touchscreen operation

- Image movement only horizontally and vertically possible using live touchscreen operation

### *Adjusting reading speed*

Tap twice to set the scrolling speed using this symbol, with which the live image runs over the display when using the scroll buttons:

- low reading speed

- medium reading speed

- high reading speed

If you tap this symbol × twice, this setting will be accepted and an example of the reading speed will be shown in the background.

### *Calling up the quick guide*

Tap twice and then tap the language symbol several times in order to select the required language. You can use  /  to scroll the preview forwards or backwards. Operation is similar to viewing images from the SD card. To exit full-screen mode while the display is zoomed out, press the   button.

### *Calling up the second menu level*

Tap × twice to call up the second menu level.

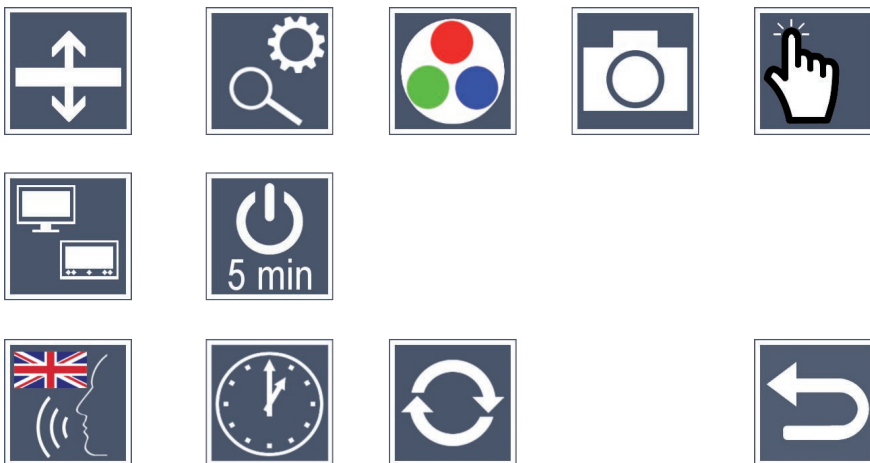
### *Quit the menu*

Tap twice to return to live build display.

## Extended settings

*Setting the reading guideline/line*

Tap twice to set further details using these symbols:

- Tap to switch between "red reading guideline" or "reading line"

- Tap repeatedly to move the reading guideline upwards or to magnify the reading line

- Tap repeatedly to move the reading guideline downwards or to shrink the reading line

*Setting maximum zoom*

Tap twice and then tap twice on the symbol for the zoom factor which should be available as a maximum.

*Adapting false colours*

Tap twice and then twice on the false colour combination which should be available later on when pressing the  button . Black and white plus inversion are activated by default. Non-activated combinations are shown crossed out.

### *Switching the photo button function on and off*

Tap twice to deactivate the photo button function.

EN

### *Switching the touch function on and off*

Tap twice to deactivate the touch function during live image display. Touch operation is retained in the menus.

### *Switching the dual screen function on/off*

The symbol for the current dual screen function setting is displayed. This allows you to determine whether the built-in display will continue to be displayed if an HDMI device is connected. Tap the symbol repeatedly to select the following settings:

- dual screen function switched off

- dual screen function switched on

### *Changing the timeout settings*

As standard, the device switches off automatically after approx. 5 minutes if it is not used.

Tap twice on  to extend this period to approx. eight hours (during mains operation).

### *Selecting the language for language output*

The symbol for the current language output setting is displayed. If necessary, tap the symbol repeatedly to select one of the available languages: or select  or  for a signal tone confirmation.

While loading the language file the  symbol appears.

### *Setting the date and time*

Tap twice to set the date and time. Tap the number you want to set. Tap the  or  symbol repeatedly to increase or decrease the number. Tap the  symbol to select the next setting.

"24H" or "12H" stand for the 24 or 12 hour display:

24H - 24-hour display selected

12H - 12-hour display selected

## Updating the firmware

Tap twice to display the current device firmware version and to update it if necessary.

Devices registered to the user will be informed about available firmware updates.

EN

**Tip:** Have the firmware updated by your optician or your Eschenbach specialist dealer!

**Never switch the device off and never remove the SD card from the device during an update.** Otherwise, the device could be irreparably damaged. This type of damage is not covered by the guarantee! Power the device from the mains using the mains adapter. This ensures that the update does not fail due to a lack of power supply.

To update the firmware please proceed as follows:

1. Connect the device USB connection  to a PC using the attached USB cable. Ensure that the SD card  is located in the device card slot.
2. The contents of the SD card will now be displayed on the PC as a drive.
3. Use that a connected PC to open the web address which was notified to you in the email concerning the update.
4. Download the update and save the "VD12FW.BRN" file directly to the SD card.
5. Select the "Safely remove hardware and eject media" function on the PC - you can find the symbol for this function in the operating system taskbar. Then disconnect the connection to the PC.
6. To start the update, go to the "Update firmware" menu and tap  twice. If you would prefer to cancel the update and return to the previous page, tap  twice. A message will be shown on the display during the update.

The device switches itself off automatically after a successful update.

## Connections

### HDMI input

You can use the enclosed HDMI cable to connect the device to other devices, using its HDMI input ⑦.

If the device detects the HDMI signal at the input, it automatically switches to display this signal. To switch back to the camera picture, press and hold the MENU button ⑫ for 3 seconds. To change back to the HDMI signal, press and hold the MENU button ⑫ again for 3 seconds.

If the connection to the HDMI device is interrupted, the device automatically switches to the camera image.

### HDMI output

You can use the HDMI cable supplied to connect the device from its HDMI output ⑰ to the HDMI input on another device, e.g. a television. This enables both the camera image and the image files to be displayed on the connected device.

### USB connection

You can use the USB cable supplied to connect the device from its USB connection ⑬ to the USB connection on another device, e.g. a computer. The SD card inserted in the device will then be made available as the "ESCHENBACH" drive for data operations.

In addition, the device can be supplied with power and charged via this connection (included power supply or USB power supply, for example from a computer).

## Technical data

Display size:	29.5 cm (11,6")   16:9   Full HD
Zoom range:	2× – 22×, also infinitely variable
EN Display modes:	real colour, contrast-enhanced up to 14 false colour combinations selectable
Operating time:	approx. 2:30 hours
Charging time:	approx. 4:30 hours
Card formats supported:	SD, SDHC (max. 32 GB)
Power supply charging device:	100 V–240 V~   50–60 Hz
Battery:	lithium-polymer
USB connection:	Type C
Temperature range:	+10 °C to +40 °C
Dimensions (W × H × D):	290 × 200 × 40 mm
Weight:	1050 g

This manual is also available on the Internet on <http://www.eschenbach-optik.com/>

USB® is a registered trademark of USB Implementers Forum, Inc.

HDMI and the HDMI logo are trademarks or registered trademarks of the HDMI Licensing LLC in the USA and other countries.

## Reset

If the device stops working as described in this manual, it may be due a fault that you can rectify yourself by resetting the device. Then press the RESET button ⑳.

## Care instructions

- ▶ When cleaning the device do not use any soap solutions that contain softening agents, no alcohol-based or organic solvents and no abrasive detergents!
- ▶ Do not clean the device in an ultrasonic bath or under flowing water!
- ▶ Only use a soft, dry cloth to clean the device. For heavier soiling (e.g. fingerprints), moisten the cloth slightly.

## Disposal

- ▶ When the reader reaches the end of its useful life, do not dispose of it with the normal household waste. This product is subject to the European directive 2012/19/EU.
- ▶ Dispose of the product or parts of the product via an approved disposal facility or your municipal disposal facility. Please comply with all applicable regulations. In case of doubt, contact your disposal facility. Dispose of all packaging materials in an environmentally responsible manner.

## Warranty

Within the parameters of the applicable statutory provisions, we provide a warranty for the functioning of the product described in this manual with regard to faults arising in connection with manufacturing defects or material defects. No warranty claims can be accepted for damage that arises due to improper handling, including damage from falls or impacts. Warranty claims can only be accepted on presentation of the sale receipt!

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USB® is een wettig gedeponeerd handelsmerk van USB Implementers Forum, Inc.
Eschenbach Optik GmbH